

Conversor Pro

The Clarity of Sound

Assistive Listening Devices



Easy Listening

Anywhere – for active lifestyles

Introducing The Conversor® Pro – an affordable assistive listening device that offers users the versatility to enjoy the most out of everyday life.

With unmatched comfort, ease of use, and wireless portability, The Conversor® Pro is the perfect companion for anyone with a hearing difficulty. It is designed for easy use with Telecoil-equipped hearing aids, cochlear implants, and binaural headphones.

The Conversor® Pro focuses on desired sounds, near or far away, filtering out unwanted background noise and greatly enhancing sound level and clarity. By removing background noise, the desired sound is much clearer and more intelligible, even in noisy environments. And by increasing the available sound level, less strain is placed on hearing aid amplifiers, eliminating distortion and feedback.

With The Conversor® Pro, users can now take a more active part in everyday listening situations, without the frustration of not hearing properly. It enables easy listening and relaxed conversation anywhere users happen to be – whether in the classroom or office, at social gatherings, driving in the car, out shopping, or at home. While The Conversor® Pro is able to meet the special listening requirements of hearing impaired users in nearly all environments, it greatly reduces the need for other assistive listening devices, such as expensive infrared or induction loop systems.

Designed for comfortable and discreet use

The Conversor® Pro consists of two lightweight components – a microphone/transmitter and a pendant receiver.


The compact, wireless Microphone/Transmitter provides enhanced sound level and clarity, near or far away from the user. Users can focus on the sounds they want to hear by selecting their desired mode of hearing – whether the Zoom setting for focused hearing and one-to-one conversations or the Omni-Directional setting for group discussions and social gatherings.

The Microphone/Transmitter can be held in the hand and pointed directly at the desired sound source; placed on a suitable surface in close proximity to the sound source up to 25 metres away; or worn by a speaker moving about the house, or in lecture and classroom situations. It can also be connected directly to a TV, stereo, VCR/DVD/CD player, or computer, for enhanced hearing enjoyment.

The Pendant Receiver is worn by the user around the neck, either above or discreetly beneath clothing, and sends the much improved sound received from the transmitter to either a hearing aid, cochlear implant, or binaural headphones, which are easily plugged into the receiver.

A local microphone on the Pendant Receiver enables the user to operate it as an independent Assistive Listening Device for amplification of one-to-one conversations.

Both the Microphone/Transmitter and Pendant Receiver are quickly rechargeable for up to eight hours of continuous use.

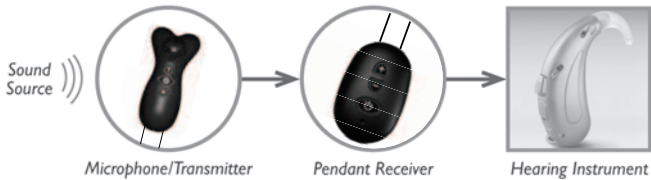

How is The Conversor® Pro different?

The Conversor® Pro provides hearing impaired users with optimal sound quality and greater versatility in a wide variety of listening situations. This is made possible by a unique set of features:

Background Noise Reduction. The Conversor® Pro – reduces unwanted background and peripheral sound by up to 30 dB, emphasizing the selected sound and allowing users to focus clearly on what they want to hear.

Wireless Transmission. Allows flexibility at greater distances. The microphone can be placed near the sound source, transmitting back to the user with enhanced clarity up to 50 metres away.

Zoom Setting. Allows the user to focus on one particular sound amongst a number of sounds, Other background or peripheral sounds are greatly reduced.

Omni-Directional Setting. Picks up nearby sounds clearly from all around the user – ideal for family dinners or meetings. For example, placing the microphone on the table allows easy listening amongst a conversing group.

Group Settings. The Conversor® Pro Microphone/Transmitter and multiple Pendant Receivers can be used for groups of hearing impaired users, either wearing headphones or hearing aids. Where the speaker wears the Microphone/Transmitter the sound will carry up to 50 metres to the wearers of the Pendant Receivers indoors, and considerably greater distances outdoors. The Conversor® Pro is also available in Multipacks for schools, colleges, and groups. Ask your distributor for more details.



Improved listening in a variety of situations...



Busy, noisy environments. The Conversor® Pro allows the user to focus on the desired sound source. Its directional microphone greatly reduces unwanted sounds coming from the sides and rear.

Quieter sounds. The Conversor® Pro automatically amplifies desired sounds, allowing users to clearly hear even quieter sounds without straining or having to turn up their hearing aid.


Relaxed conversations. The Conversor® Pro provides greatly enhanced sound level and clarity, allowing users to interact with family, friends or work colleagues in a more relaxed manner.



In a Classroom/
Lecture Hall



In a Busy
Restaurant/Café

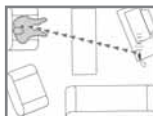

Watching
Television



Driving in
the Car

*Enhanced sound quality
no matter where you
happen to be...*



Conversor Limited

The Lansbury Estate,
102, Lower Guildford Road,
Knaphill, Woking, Surrey,
GU21 2EP (UK)

T: +44 (0) 870 066 3499
Freephone: 0800 298 9982 (UK only)
F: +44 (0) 870 066 3669
E: info@conversorproducts.com
W: www.conversorproducts.com